

## Class II Appliances: Herbst®, MARA, X-Bow®

**Models/Scans:** Upper and lower models in Class I or Class II stone (models). Complete anterior and posterior anatomy of upper and lower arches captured. Spacers can be placed after the impression/scan is taken.

**\*When palatal accessories are being requested (expansion mechanisms) be sure that the palatal anatomy is also captured in the scan/model.**

**Wax Bites:** Herbst® mechanisms are positioned for an end-to-end relationship on Class II and Division II patients. Division I patients are positioned in a Class I molar relationship. Wax bites or lines on models will override standard positioning. Patients are not advanced more than 6 mm unless requested.

**Notes:** Specialty can supply and fit crowns, bands and the new ROCs (removed occlusal crowns) indirectly when anchorage teeth are fully erupted.

**\*Please evaluate the eruption of teeth selected for anchorage. If anchorage teeth are not fully erupted, please do either of the following: (1) Fit bands/crowns and send loose in a bag (or in impression) for technicians to fit directly or (2) consider other teeth for anchorage.**

